Manual de usuario del Lomo Litio V.11/09

Stricker
HAND-Bikes

R & E Stricker GmbH
Reha-Entwicklungen
Klotzberg 64
D-77815 Bühl/Baden
Tel. 07223/72510
Fax. 07223/74947
www.stricker-handbikes.de
E-Mail:
info@stricker-handbikes.de

RODEM ORTOTEC S.L.
C/ San Juan de Dios, 3
46011 Valencia
tel.- 96 110 00 74
tel./fax- 96 355 64 15
info@rodem.es
www.rodem.es

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Warning:

1. Points should be noted when using this battery pack with your electric bicycle. If the below guidelines are not being strictly followed, the running time will be lessened. And at the same time, the usage life of the battery pack and your electric bicycle may also be shortened.
 - I. Avoid over-loading the electric bicycle for over 75kg.
 - II. Avoid starting run the bicycle from its motionless state by using electric power..
 - III. When climbing uphill, human assist should be applied.
 - IV. Avoid extended climbing slope of over 15° steep.*
 - V. If riding on poor road surface, human pedaling is recommended.
2. If the battery is found overheating, leaking, deforming, giving out odd smell and smoked, stop using the battery pack immediately. Put it in open area and away from the crowd.
3. The battery pack is designed form using with the designed electric bicycle. Do not use the battery pack on other applications.
4. Use only the specified adaptor to charge the battery. It is prohibited to use adaptor not certified by our company for charging to avoid damaging the battery pack.
5. Do not place the battery pack over the radiator or any heated objects, particularly microwave oven and high-pressure container, to avoid the heat from damaging the battery pack. It is also important that the keep the battery pack way from reach by children.
6. For extended storage of the battery pack, it is recommended that you charge the battery with 30% – 50% capacity, i.e. with 2-3 LEDs on. Repeat the same procedure every 4 months. Totally discharged battery should be recharged immediately afterwards.
7. If the battery pack or adaptor is found defective, please return for repair immediately. Please do not attempt to open the casing of the battery pack and the adaptor to avoid hazard or damaging the parts inside. Warranty will be voided if the battery pack or the adaptor is found being opened.

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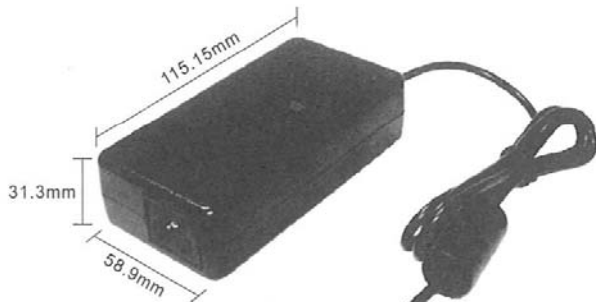
ELECTRIC BICYCLE BATTERY CHARGER USER MANUAL

EN

COMPOSITION

001

002

003

005


4 output plugs are available, can be used in different charging socket

SPECIFICATIONS

Model	ACK4201	ACK4211
Application	For 36V Series Li-ion Battery	For 36V Series LiFe Battery
Input	100-240V~50/60Hz 1.2A (MAX)	
Output Voltage	42.3VDC \pm 0.5V	38.5VDC \pm 0.5V
Output Current	1.35A \pm 0.05A	
Output Power	Max. 60W	
Color	Black	
Measurement	115.15 x 58.9 x 31.3 mm	
	Length of DC Output Cord: 1M	
Weight	Approx. 280g	

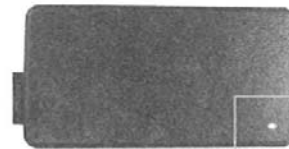LED INDICATOR

STATUS:

RED: Charging
GREEN: Standby / Fully Charged

The LED Indicator will show the status of the Battery Charger.

POWER CORDS

We have 5 types of AC Power Cord for choice:

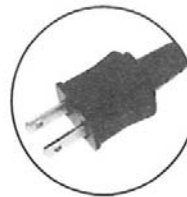
Australian

European

Britannic

Japanese

American

CAUTION

- Do not attempt to open or modify this unit. All work should be carried out by authorized service personnel only.
- Use only with compatible batteries. Charging non-compatible batteries may cause fire, electrical shock or other accidents.
- When connecting to the battery, be sure the DC output plug is connected to the charging socket firmly.
- During charging, the temperature of the charger will rise, this is normal and not a malfunction.
- A short circuit between the plus (+) and minus (-) charge pins may cause fire, electrical shock or other accidents. Make sure that the charge connector is isolated from any metal parts when a battery is not connected.
- Only use the AC power cord included with the charger or the ones specified.
- Stop charging immediately if charging does not complete in designated time. In case of abnormal smell, leak or case deformity during use, turn off the power and unplug the cable from socket to avoid possible injury.

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
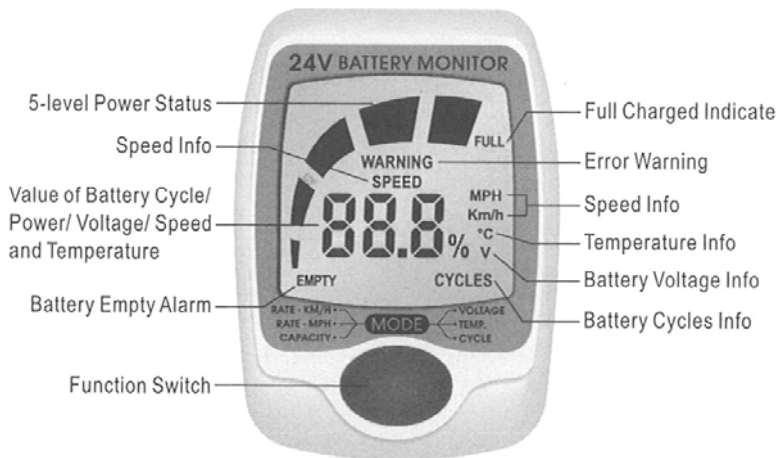
Electric Bicycle Intelligent LCD Monitor Instruction Manual

EN

COMPOSITION


INTERFACE DESCRIPTION

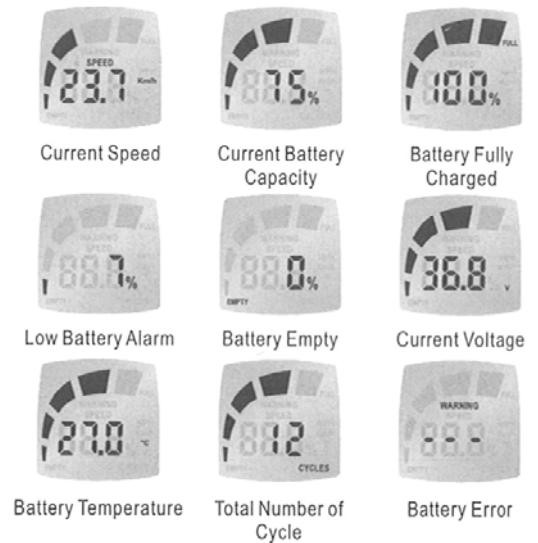CABLE CONNECTION

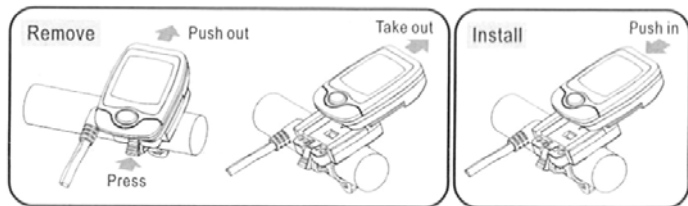INSTALL AND REMOVE

Remove - Press the button, and push the LCD Instrument panel outward to disassemble it from its base

Install - Push the LCD Instrument panel inward to its base, until the panel buckles itself into the base

ERROR

- When the LCD Instrument display is connected to the power supply, it does not show any information. Please check the cable connections to see whether they are wrongly connected or loosened.
- The " - - - " showing on the LCD indicates signal communication failure, please check the connection of SDA (brown colour) and SCL (Orange) to see whether they are connected reversely, short-circuited, loosened or broken.
- When there is any problem in displaying the correct voltage, please reinstall the LCD Instrument panel to its base.

FUNCTION

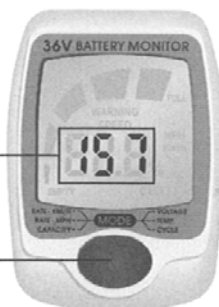
The LCD Instrument panel is compact, user-friendly. Different parameters, including total number of cycles, current battery capacity, current battery voltage, current battery temperature, current speed etc, can be checked simply by using a button. In addition, the wheel circumference can be set to match up its use on different bicycles.

- After the LCD instrument panel is connected to the power supply, its backlight will flash and display the battery data.
- Press the function button (less than 1 second) to switch 6 interfaces: total number of cycles → current battery capacity → current battery voltage → current battery temperature → current speed (Km/h) → current speed (MPH)
- LCD backlight lights up for 5 seconds when the function button is pressed.
- When the LCD instrument panel detects that it is not communicated with the battery, it will display "WARNING" and "" etc.

Circumference Unit is cm

Press 3 sec. Circumference setting interface

Press 1 sec. Modify the wheel circumference data digit by digit

Wheel circumference setting instruction:

Press function button for 3 seconds to the wheel circumference setting interface, LCD instrument panel will display current wheel circumference, its unit is cm, and its maximum value is 255cm.

The wheel circumference data should be modified digit by digit, from left to right. The modifying digit keeps flashing, and its value is added by 1 by pressing the function button each time. When the suitable digit is set, please stop pressing and wait for 5 seconds to modify the next digit. Please repeat the above steps for the remaining 2 digits. After the modification of the last digit, the interface will automatically change to the battery status interface.

Note:

Since the wheel circumference is set irreversibly from left to right. If there is any digit which is wrongly modified, it cannot be changed until reentering the interface.

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